



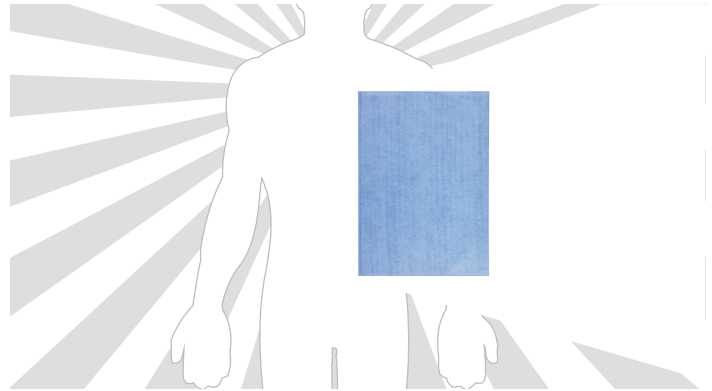

Scatter Armor Multipurpose Shield

SA-MPS, SA-MPS-CS

Transradial Access: Left and Right Radial, Shunt, Biliary Drainage, Spinal, Fibroid Embolization

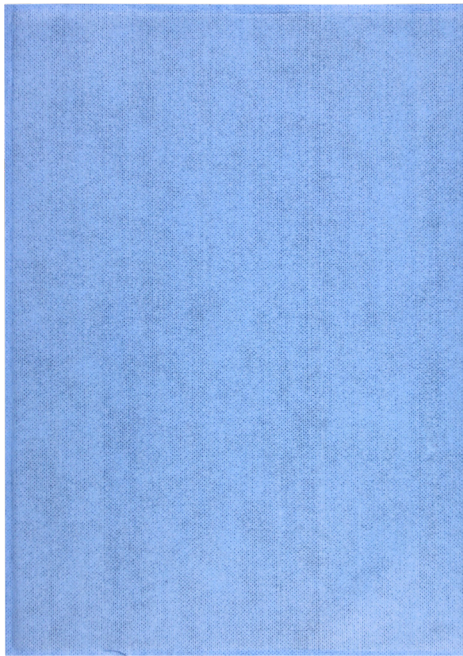

Model Number	Description
SA-MPS	Scatter Armor Multipurpose Shield, Absorbent, No Fenestration Box of 15
SA-MPS-CS	Scatter Armor Multipurpose Shield, Absorbent, No Fenestration Case of 45
Dimensions	12" x 17"
Attenuation	Greater than 92% at 90 kVp
Protection	Greater than 0.25mm LE
Core Material	KIARMOR Bi-Layer Lead-Free



Single Use Only

STERILE EO



Infab Corporation
1040 Avenida Acaso
Camarillo, CA 93012
E-mail info@infabcorp.com
Call: 805-987-5255
Fax: 805-482-8424



Do Not Use if Product is Damaged Or Compromised

WARNINGS

Product is sterile unless package has been opened or damaged. To be used in addition to, not in place of, routine radiation protection devices.

Consult Instructions for Use

1. Remove drape/shield from the pouch without compromising sterility.
2. Place drape/shield on the patient
3. Keep the drape/shield out of the direct beam
4. Position the drape/shield on the patient so that it lies between direct beam and the doctor or technician to maximize the safety shadow.
5. Secure the drape/shield in the proper position on the patient drape (using the adhesive strip on drape/shield). Do not place adhesive directly on patient's skin